

## What You Need

To use the Digital Diagnostics assessment system, you need a PC with Microsoft Windows, internet access, and a working printer.



(More specifically, you need Windows 98 or later, Internet Explorer 5.0 or later, and an available COM port. You should be able to surf the web and print from your computer.)

## What You Get

**1. Handheld tablet.** You will receive a handheld computer tablet that resembles an Etch-a-Sketch. These rugged computer tablets have video display screens and numeric keypads. They are wireless and portable.



Patients self-administer the assessment tools using the handheld tablet. They see questions on the video screen and respond using the keypad.

**2. Docking station.** You will also receive a docking station (not shown). It connects to your PC and lets the handheld tablet communicate with your PC.

## How it works



**1. Administer the test.** Patients self-administer diagnostic tests using the handheld computer tablet. They can do this in the waiting room or any other convenient location. The handheld computer tablet stores the responses.

**2. Dock the tablet.** When the patient finishes, place the handheld computer tablet on the docking station.

**3. Behind the scenes.** The patient's responses are transferred instantly to our server via the internet and recorded in a secure (HIPAA compliant) database. Meanwhile, the test is scored automatically and the diagnostic report is sent to your printer. All this happens invisibly, in the background. All you see on your end is...




**4. Immediate, real-time Results.** Within seconds, the report prints on your printer. That's all there is to it.

**There are no forms to score, no responses to tabulate, and no paperwork. You immediately get a chart-ready report, printed in real-time on your office printer.**

## Diagnostic Report with Trending

The test reports are chart ready and very user friendly. For example, the QPD Panel report resembles a standard blood chemistry lab report, with a “look and feel” familiar to all physicians.

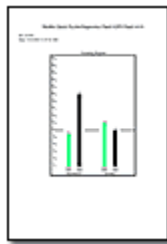

When a patient takes the test more than once, the report includes a trending graph showing changes over time. Use the patient trending graph to monitor patient progress or assess treatment outcome.

For more information call us at 800.559.9885 or visit us on the web at [www.DigitalDiagnostics.com](http://www.DigitalDiagnostics.com)